







RELIWASH

# Eye Emergency Plan; which product to use?

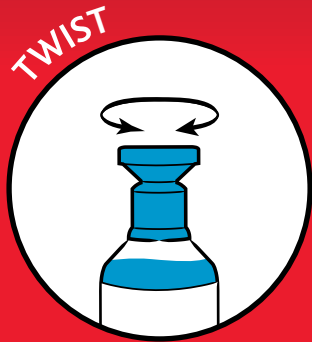

	CONTAMINANT	ACTION	DIRECTIONS
	Dirt, dust, metal, wood, splinters.	To rinse the eyes and prevent foreign substances coming into contact with them and causing damage.	Rinse until the substance is removed, or medical help arrives.
	Acids. (e.g.) <i>Hydrochloric Acid</i>	To quickly neutralise the pH value and prevent corrosive damage to the eyes and skin.	Rinse with Redcap™ Buffer Solution. Continue rinsing with saline eye wash or water until medical help arrives.
	Alkalis. (e.g.) <i>Sodium Hydroxide</i>	To quickly neutralise the pH value and prevent corrosive damage to the eyes and skin.	Rinse with Redcap™ Buffer Solution. Continue rinsing with saline eye wash or water until medical help arrives.
	Oil and solvents.	To rinse the eyes until the unpleasant sensation is over.	Continue rinsing until you receive medical attention.
	Skin Damage. <i>Caused by acids, alkali or chemicals.</i>	To quickly neutralise the pH value and prevent corrosive damage to the eyes and skin.	Rinse with Redcap™ Buffer Solution. Continue rinsing with saline eye wash or water until medical help arrives.

